



Quick Reference How to:

- Change your password – From the Studies Tab click on Option then My Profile and Enter your new password and then enter it again in the confirm password box. Scroll to the bottom and click save.
- Access a report – click the report button on the right side of the viewer.
- Print a report – click the report button on the right side of the viewer and then click the print button.
- Download a report - click the report button on the right side of the viewer and then click the PDF button  (submenu item under the printer button).
- Access Key images (if any exist) – Key Images will be the first series/thumbnail. Drag this to a View Port to view.
- Download a study – From the Search results chose the study to download by clicking on it then click the Transfer button at the bottom of the page. On the window that opens choose local system (if not already selected) and click transfer. This will download a ZIP file containing the study. You will need to open and drag the folder(s) out. Once done you can use this to import into your system.
- View Connection quality – The connection quality is by default on the login page. If you would like to have it visible after logging in then go to Options a User Preferences and put a check in the Enable Connection Test Tool.  This will enable the connection tool on the Studies Tab when the user is NOT viewing a study.
- How do I access Marquis' iConnect Access?

URL: <https://iconnect.marquisdagnosticimaging.com>
 Existing Users: Use your login and password
 NEW USERS: Click on Sign Up



| |
|-----------|
| Username: |
| Password: |